

# Instructions for use



# Bure<sup>TM</sup> Walker

**GATE**

REHAB DEVELOPMENT AB

Bure Standard GAS 56-311

Bure S GAS 56-301

Bure Low GAS 56-322

Bure Low S GAS 56-322-S

## Dear User,

*Congratulations on obtaining a Bure Walker! These instructions provide important information on how to use the walker and how to maintain it.*

*All of us here at GATE Rehab Development hope you enjoy using your new product. If you have any comments, questions or points of view, please contact us on +46 (0)371 318 00.*



### Raising the frame

Release the gas strut by moving the handle upward (the gas strut will raise the walker). Let go of the handle at the desired height (the gas strut will lock).

### Lowering the frame

Release the gas strut by moving the handle upward. Use your body weight to press the frame down. Let go of the handle at the desired height (the gas strut will lock).



#### NOTE!

The handle may not be pushed down. There is a risk of breaking it.




## Adjusting the handles

To adjust handles to the desired position; loosen screw, adjust handle, tighten screw.



## Left and right adjustment of armrests

The armrests can be adjusted sideways for optimal support; loosen screw, adjust armrest, tighten screw.

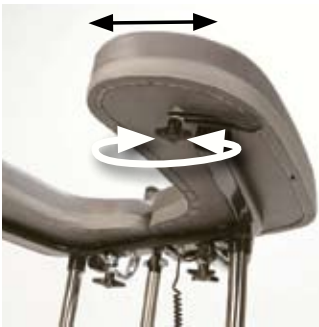

## Locking castors

Depress the foot brake to set it and lock the castor.



## Unlocking castors

Press down on the protruding part of the brake to release it.



# Maintenance

Recommended at least once per year.

Check that	Action
<b>General</b> The walker feels firm/free from play.  The walker does not rattle during manoeuvres.  The walker is level and all of its castors are in contact with the floor.  The walker is not dirty.	Clean with lukewarm soap solution or alcohol-based cleaning agent (no petroleum products).
<b>Armrests</b> Armrests are intact and clean. Armrest widening is functional.	Clean; replace if damaged. Fit lock screws/plates as required.
<b>Handle</b> Grips are not damaged/dirty.  Handle adjustment is functional.	Clean with lukewarm soap solution (no petroleum products). Replace handle grips. Fit lock screws/plates as required.
<b>Height adjustment</b> Raising and lowering are functional.  The walker feels stable at maximum settings.  There is no play between the vertical frame and bottom frame.  Height adjustment clamps lock.  The gas strut fitting must be free from play.  Height adjustment by means of the handle is working. The gas strut must slide freely without squeaking.	Fit new guide sleeves (in frame) or end plugs (in chrome tubes) as required.  Tighten all lock bolts.  Replace clamps.  Tighten the attachment fitting concerned; replace bolts/lock nuts as necessary.  Replace gas strut, handle.
<b>Frame parts</b> There is no mechanical damage.  There are no scratches. End plugs/lower frame fitted.	If damage is present, contact GATE's customer services.  Touch up as necessary. Fit new end plugs.
<b>Castors/brakes</b> Castors roll easily/tread not damaged.  The castors are firmly fastened to the lower frame.  Castor brakes functional on all castors.  Handbrake functional.	Clean or replace castors. NOTE! Castors are always fixed to the lower frame with thread locker or lock nuts. Do not dismantle the castors; replace the whole castor assembly instead.  Tighten the castor bolt and fit locking nut or use thread locker (depends on model).  Replace castor assembly.  Adjust the brake or fit new castors.

## General maintenance advice

- Clean the walker and its armrests clean as necessary. Wipe using a soap solution/surface disinfectant.
- The product can be washed in a pressure washer/machine at 60°C for ten minutes. To increase lifetime, avoid aiming high pressure water jets at bearings.
- Replace damaged parts immediately; must be done by experienced personnel.
- The product is only intended for indoor use.

## Technical specifications

Dimensions apply to walkers with 125 mm castors.

Model	Part No.	Width	Max user weight	Length	Height	Armrest/ internal dimension
Standard GAS	56-311	78 cm	150 kg	80 cm	90-129 cm	25-50 cm
S GAS	56-301	66 cm	150 kg	79 cm	90-129 cm	30-45 cm
Bure Low GAS	56-322	78 cm	150 kg	80 cm	75-95 cm	25-50 cm
Bure Low S GAS	56-322-S	66 cm	150 kg	79 cm	75-95 cm	30-45 cm



CE marked according to MDD 93/42/EEC.  
Tested and approved according to EN-11199-3/2005



**WE DO BETTER.**

Would you like to know more?

Contact us at:  
Tel: +46 371 31800  
Gate Rehab Development AB  
Industrigatan 2 | SE-333 75 Reftele  
info@gaterd.com | www.gaterd.com



GATE COLLECTION